

AEROMONAS SELECTIVE SUPPLEMENT**FOR 500 ml OF AEROMONAS AGAR**

Freeze-dried mixture for the isolation of *Aeromonas* spp.

Description	
Code number:	4 ml: AES80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction: Suspend **30 g of Aeromonas Agar Base (AEA20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Cool to 50 °C and add aseptically the contents of **one vial of Aeromonas Selective Supplement (AES80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

Warning!

The medium is heat sensitive.
No further sterilisation is necessary or desirable.

FORMULA [mg/vial]

Ampicillin	2,5
------------	-----

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!