

## AMPICILLIN (5 mg) SUPPLEMENT

FOR 500 ml OF COLUMBIA BLOOD AGAR + AMPICILLIN

Freeze-dried mixture for the supplementation of Columbia Blood Agar Base.

<b>Description</b>	
Code number:	4 ml: AMP80004
Colour:	White
Appearance:	Homogeneous lyophilisate

**Direction:** Suspend **21 g of Columbia Blood Agar Base (COL20500)** in 475 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically **25 ml of sterile defibrinated sheep blood**. Dissolve the contents of **one vial of Ampicillin (5 mg) Supplement (AMP80004)** with 4 ml of sterile distilled water and add aseptically to the Columbia Blood Agar at 50 °C. Mix well before pouring.

### FORMULA [mg/vial]

Ampicillin	5
------------	---

**Storage conditions:** Protected from light, at 2-8 °C.

**Warning!**

Use before the expiry date on the label.

**In vitro diagnostic – for professional use only!**