

BORDETELLA SELECTIVE SUPPLEMENT

FOR 500 ml OF BORDETELLA AGAR

Freeze-dried mixture for the isolation of *Bordetella* spp.

Description	
Code number:	4 ml: BSS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction: Suspend **26 g of Charcoal Agar Base (CHA20500)** in 450 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically **50 ml of sterile defibrinated blood** and the contents of **one vial of Bordetella Selective Supplement (BSS80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

FORMULA [mg/vial]

Cephalexin	20
------------	----

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!