

CAMPYLOBACTER GROWTH SUPPLEMENT

FOR 500 ml OF CAMPYLOBACTER AGAR

Freeze-dried mixture for the enhanced growth of *Campylobacter* spp.

Description	
Code number:	4 ml: CGS80004
Colour:	Yellow
Appearance:	Homogeneous lyophilisate

Direction: Suspend **19 g of Campylobacter Agar Base (CAA20500)** in 470 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically **25 ml of sterile lysed horse blood** and the contents of **one vial of Campylobacter Growth Supplement (CGS80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

FORMULA [mg/vial]

Sodium pyruvate	125
Sodium metabisulphite	125
Ferrous sulphate	125

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!