

CAMPYLOBACTER SELECTIVE SUPPLEMENT, BLASER-WANG

FOR 500 ml OF CAMPYLOBACTER SELECTIVE AGAR,
BLASER-WANG

Freeze-dried mixture for the isolation of *Campylobacter* spp.

Description	
Code number:	4 ml: CBW80004
Colour:	Yellowish
Appearance:	Homogeneous lyophilisate

Direction: Suspend **19 g of Campylobacter Agar Base (CAA20500)** in 470 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically **25 ml of sterile lysed horse blood** and the contents of **one vial of Campylobacter Growth Supplement (CGS80004)** reconstituted with 4 ml of sterile distilled water. Dissolve the contents of **one vial of Campylobacter Selective Supplement, Blaser-Wang (CBW80004)** with 4 ml of sterile distilled water and add aseptically to the above. Mix well before pouring.

FORMULA [mg/vial]

Cefalotin	7,5
Vancomycin	5,0
Trimethoprim	2,5
Amphotericin B	1,0
Polymyxin B	0,2

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!