

CAMPYLOBACTER SELECTIVE SUPPLEMENT, CAT

FOR 500 ml OF CAMPYLOBACTER BLOOD-FREE (CAT) AGAR

Freeze-dried mixture for the isolation of *Campylobacter* spp.

Description	
Code number:	4 ml: CCP80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction: Suspend **24 g of Campylobacter Blood-Free (CCDA) Agar Base (CCA20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Campylobacter Selective Supplement, CAT (CCP80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

FORMULA [mg/vial]

Amphotericin B	5
Cefoperazone	4
Teicoplanin	2

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!