

CAMPYLOBACTER SELECTIVE SUPPLEMENT, KARMALI

FOR 500 ml OF CAMPYLOBACTER AGAR, KARMALI

Freeze-dried mixture for the isolation of *Campylobacter* spp.

Description	
Code number:	4 ml: CPK80004
Colour:	Yellowish
Appearance:	Homogeneous lyophilisate

Direction: Suspend **23 g of Campylobacter Agar Base, Karmali (CAK20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Campylobacter Selective Supplement, Karmali (CPK80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

FORMULA [mg/vial]

Cycloheximide	50
Sodium pyruvate	50
Cefoperazone	16
Vancomycin	10

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!