

**CAMPYLOBACTER SELECTIVE SUPPLEMENT, PRESTON****FOR 500 ml OF CAMPYLOBACTER SELECTIVE AGAR, PRESTON**

Freeze-dried mixture for the isolation of *Campylobacter* spp.

Description	
Code number:	4 ml: CPS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

**Direction:** Suspend **19 g of Campylobacter Agar Base (CAA20500)** in 470 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically **25 ml of sterile lysed horse blood** and the contents of **one vial of Campylobacter Growth Supplement (CGS80004)** reconstituted with 4 ml of sterile distilled water. Dissolve the contents of **one vial of Campylobacter Selective Supplement, Preston (CPS80004)** with 4 ml of 1 : 1 mixture of acetone and sterile distilled water and add aseptically to the above. Mix well before pouring.

**FORMULA [mg/vial]**

Cycloheximide	50,0
Rifampicin	5,0
Trimethoprim	5,0
Polymyxin B	0,4

**Storage conditions:** Protected from light, at 2-8 °C.

**Warning!**

Use before the expiry date on the label.

**In vitro diagnostic – for professional use only!**