

CEFIXIME TELLURITE SELECTIVE SUPPLEMENT

FOR 500 ml OF MACCONKEY AGAR, SORBITOL ES

Freeze-dried mixture for the isolation of *Escherichia coli* O157.

Description	
Code number:	4 ml: CTS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction for MacConkey Agar, Sorbitol ES: Suspend **26 g of MacConkey Agar, Sorbitol (MCS20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Cefixime Tellurite Selective Supplement (CTS80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring..

FORMULA [mg/vial]

Potassium tellurite	1,250
Cefixime	0,025

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!