

DTM SELECTIVE SUPPLEMENT**FOR 500 ml OF DTM AGAR**

Freeze-dried mixture for the isolation of dermatophytes.

Description	
Code number:	4 ml: DTS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction: Suspend **20 g of DTM Agar Base (DTM20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of DTM Selective Supplement (DTS80004)**, reconstituted with 4 ml of 1 : 1 mixture of ethanol and sterile distilled water. Mix well before pouring.

FORMULA [mg/vial]

Cycloheximide	250
Chlortetracycline	50
Gentamicin	50

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!