

FERRIC AMMONIUM CITRATE SOLUTION, STERILE

FOR 10 OR 30 LITRE OF MEDIA

A ferric ammonium citrate solution for the differentiation of *Clostridium* spp.

Description	
Code number:	10 ml: FAC80010, 30 ml: FAC80030
Colour:	Yellow
Appearance:	Transparent solution

Direction: Prepare 500 ml of sterile medium base from one of the dehydrated medium bases for the detection of *Clostridium* spp. (e.g. RCM-DRCM Medium Base, Lactose Sulphite Broth Base) according to the direction of the given medium. Cool to 50 °C and add aseptically **10 drops (0,5 ml) of Ferric Ammonium Citrate Solution, Sterile (FAC80030)** and **10 drops (0,5 ml) of Sodium Metabisulphite Solution, Sterile (SMS80030)**. Mix well before pouring.

FORMULA

Ferric ammonium citrate	60 %
Solvent	Sterile distilled water

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!