

GARDNERELLA SELECTIVE SUPPLEMENT

FOR 500 ml OF GARDNERELLA SELECTIVE AGAR

Freeze-dried mixture for the isolation of *Gardnerella vaginalis*.

Description	
Code number:	4 ml: GAS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction: Suspend **21 g of Columbia Blood Agar Base (COL20500)** in 475 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically **25 ml of sterile defibrinated sheep blood**. Mix well. Dissolve the contents of **one vial of Gardnerella Selective Supplement (GAS80004)** with 4 ml of sterile distilled water and add aseptically to the Columbia Blood Agar at 50 °C. Mix well before pouring.

FORMULA [mg/vial]

Nalidixic acid	15
Gentamicin	2
Amphotericin B	1

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!