

HAEMOPHILUS SUPPLEMENT**FOR 500 ml OF HAEMOPHILUS TEST AGAR**

Freeze-dried mixture for the isolation of *Haemophilus influenzae*.

Description	
Code number:	4 ml: HTS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction: Suspend **21,5 g of Haemophilus Test Agar Base (HTM20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Haemophilus Supplement (HTS80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

FORMULA [mg/vial]

Nicotinamide adenine dinucleotide (NAD)	7,5
Hemin	7,5

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!