

LOEFFLER'S METHYLENE BLUE STAIN

Staining solution for the bacteriological and histological investigation of sample material of human origin.

Description	
Code number:	1000 ml: LME61000
Colour:	Blue
Appearance:	Dark, precipitation free solution

Procedure:

1. After a careful fixation flood the slide with the stain. Keep on the slide 1 – 2 minutes.
2. Rinse the slide thoroughly with water, dry on air.

Evaluation:

Micro-organisms and white blood cells: dark blue

Background: grey

FORMULA

Methylene blue	3 g/l
Solvent	Ethanol 30 %

Storage conditions: Store the bottles protected from light at room temperature. Use before the expiry date on the label.

Quality control:

Test strains	Results
<i>Escherichia coli</i>	Blue colouration

In vitro diagnostics – for professional use only!