

# MRSA SELECTIVE SUPPLEMENT

FOR 500 ml OF MRSA SCREEN AGAR

Freeze-dried mixture for the detection of MRSA.

<b>Description</b>	
Code number:	4 ml: MSS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

**Direction:** Suspend **39 g of MRSA Screen Agar Base (MRS20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of MRSA Selective Supplement (MSS80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

## FORMULA [mg/vial]

Oxacillin	3
-----------	---

**Storage conditions:** Protected from light, at 2-8 °C.

### Warning!

Use before the expiry date on the label.

**In vitro diagnostic – for professional use only!**