

PSEUDOMONAS SELECTIVE SUPPLEMENT, CFC

FOR 500 ml OF CETRIMIDE (CFC) AGAR No.2

Freeze-dried mixture for the isolation of *Pseudomonas aeruginosa*.

Description	
Code number:	4 ml: CFC80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction: Suspend **25 g Cetrimide Agar Base No.2 (CCT20500)** in 500 ml of distilled water. Add **5 ml of Glycerol Supplement (GLC80100)** and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the content of **one vial of Pseudomonas Selective Supplement, CFC (CFC80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

FORMULA [mg/vial]

Cefaloridine	25
Cetrimide	5
Fucidin	5

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!