

RPMI MOPS SOLUTION, STERILE

FOR 500 ml OF RPMI MOPS AGAR

Sterile RPMI MOPS solution for the preparation of RPMI MOPS Agar.

Description	
Code number:	100 ml: RGS80100
Colour:	Light orange
Appearance:	Transparent solution

Direction: Suspend **17,5 g of RPMI MOPS Agar Base (RGM20500)** in 400 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121°C for 15 minutes. Cool to 50°C. Meanwhile heat gently **100 ml of RPMI MOPS Solution, Sterile (RGS80100)** to 50 °C. Add the supplement aseptically to the agar base. Mix well before pouring.

FORMULA [g/l]

MOPS	34,5
RPMI 1640	10,5

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic - for professional use only!