

**SALMONELLA PLUS SELECTIVE SUPPLEMENT**

FOR 500 ml OF ChromoBio® SALMONELLA PLUS

Freeze-dried mixture for the isolation of *Salmonella* spp.

Description	
Code number:	4 ml: SSP80004
Colour:	White
Appearance:	Homogeneous lyophilisate

**Direction:** Suspend **22,5 g of ChromoBio® Salmonella Plus Base (SAP20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Salmonella Plus Selective Supplement (SSP80004)** reconstituted with 4 ml sterile distilled water. Mix well before pouring.

**FORMULA [mg/vial]**

Cefsulodin	6,0
Novobiocin	2,5
Specific inhibitor	1,0

**Storage conditions:** Protected from light, at 2-8 °C.

**Warning!**

Use before the expiry date on the label.

**In vitro diagnostic – for professional use only!**