

SELENITE SUPPLEMENT

FOR 500 g OF SELENITE BROTHS

Sodium selenite for preparation of selenite broths.

Description	
Code number:	110 g: SES80110
Colour:	White
Appearance:	Crystalline powder

Direction: Dissolve **4 g of Selenite Supplement (SES80110)** in one litre of distilled water. Suspend **19 g of Selenite Broth Base (SEB20500) or Selenite Cystine Broth Base, USP (SCB20500) or Selenite Cystine Mannitol Broth Base (SCM20500)** and heat gently to dissolve the medium completely. Dispense into final containers. In case the medium is not getting to use on the day of preparation, sterilise at 100 °C for 10 minutes. Cool quickly. The presence of a small amount of pinkish or brownish precipitate is not detrimental.

Warning!

The medium is heat sensitive.
No further sterilisation is necessary or desirable.

FORMULA

Sodium selenite (bacteriological grade)	110 g
---	-------

Storage conditions: Protected from light, at room temperature.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!