

SODIUM METABISULPHITE SOLUTION, STERILE

FOR THE PREPARATION OF 10 OR 30 LITRE OF MEDIA

Sodium metabisulphite solution for the differentiation of *Clostridium* spp.

Description	
Code number:	10 ml: SMS80010, 30 ml: SMS80030
Colour:	Yellow
Appearance:	Transparent solution

Direction: Prepare 500 ml of sterile medium base from one of the dehydrated medium bases for the detection of *Clostridium* spp. (e.g. RCM-DRCM Medium Base, Lactose Sulphite Medium Base etc.) according to the direction of the given medium. Cool to 50 °C and add aseptically **10 drops (0,5 ml) of Sodium Metabisulphite Solution, Sterile (SMS80030)** and **10 drops (0,5 ml) of Ferric Ammonium Citrate Solution, Sterile (FAC80030)**. Mix well before pouring.

FORMULA

Sodium metabisulphite	60 %
Solvent	Distilled water

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!