

# SULPHAMANDELATE SELECTIVE SUPPLEMENT

FOR 500 ml OF BRILLIANT GREEN AGAR, MODIFIED

Freeze-dried mixture for the isolation of *Salmonella spp.*

<b>Description</b>	
Code number:	4 ml: SUS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

**Direction:** Suspend **26,5 g of Brilliant Green Agar Base, Modified (BGM20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Cool quickly to 50 °C and add aseptically the contents of **one vial of Sulphamandelate Selective Supplement (SUS80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

**Warning!**

The medium is heat sensitive.  
No further sterilisation is necessary or desirable.

### FORMULA [mg/vial]

Sodium sulfacetamide	500
Sodium mandelate	125

**Storage conditions:** Protected from light, at 2-8 °C.

**Warning!**

Use before the expiry date on the label.

**In vitro diagnostic – for professional use only!**