

TRICHOMONAS SELECTIVE SUPPLEMENT

FOR 500 ml OF TRICHOMONAS MEDIA

Freeze-dried mixture for the cultivation of *Trichomonas vaginalis*.

Description	
Code number:	4 ml: TSS80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction for Trichomonas (CPLM) Medium, Modified: Suspend **17,5 g of Trichomonas (CPLM) Medium Base, Modified (CPL20500)** in 425 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Trichomonas Selective Supplement (TSS80004)** reconstituted with 4 ml of sterile distilled water and **70 ml of sterile inactivated (i.e. serum held at 56 °C for 30 minutes) and pH adjusted (6,0) horse serum**. Mix well and dispense aseptically into sterile test tubes.

Direction for Trichomonas Medium: Suspend **18,5 g of Trichomonas Medium Base (TRM20500)** in 455 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Trichomonas Selective Supplement (TSS80004)** reconstituted with 4 ml of sterile distilled water and **40 ml of sterile inactivated (i.e. serum held at 56 °C for 30 minutes) and pH adjusted (6,4) horse serum**. Mix well and dispense aseptically into sterile test tubes.

FORMULA [mg/vial]

Streptomycin	500
Penicillin	80

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!