

YERSINIA (CIN) SELECTIVE SUPPLEMENT

FOR 500 ml OF YERSINIA AGAR
FOR 500 ml OF YERSINIA BROTH

Freeze-dried mixture for the isolation of *Yersinia enterocolitica*.

Description	
Code number:	4 ml: CIN80004
Colour:	White
Appearance:	Homogeneous lyophilisate

Direction for Yersinia Agar: Suspend **30 g of Yersinia Agar Base (YAB20500)** in 500 ml of distilled water and heat with frequent agitation until the medium boils well. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Yersinia (CIN) Selective Supplement (CIN80004)** reconstituted with 4 ml of sterile distilled water. Mix well before pouring.

Direction for Yersinia Broth: Suspend **16,5 g of Yersinia Broth Base (YBB20500)** in 500 ml of distilled water and heat gently to dissolve the medium completely. Sterilise by autoclaving at 121 °C for 15 minutes. Cool to 50 °C and add aseptically the contents of **one vial of Yersinia (CIN) Selective Supplement (CIN80004)** reconstituted with 4 ml of sterile distilled water. Mix well and dispense aseptically into sterile test tubes.

FORMULA [mg/vial]

Cefsulodin	7,50
Irgasan	2,00
Novobiocin	1,25

Storage conditions: Protected from light, at 2-8 °C.

Warning!

Use before the expiry date on the label.

In vitro diagnostic – for professional use only!